**TESLA BREAST
PUMP BP100**
USER MANUAL

EN

DE

CZ

SK

PL

HU


SL

DESCRIPTION

1. Dust cap
2. Massage cushion
3. Pump body
4. Silicone diaphragm
5. Cap
6. Silicone tube
7. Silicone valve
8. Feeding bottle
9. Bottle cap with nipple
10. Pump motor
11. Power adaptor

MOTOR UNIT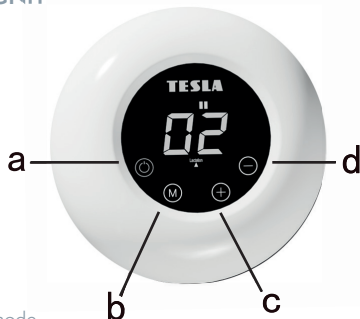

- a. On/off
- b. Change mode
- c. Increase level
- d. Decrease level

ASSEMBLY INSTRUCTIONS

Before the first use, all components of the breast pump should be cleaned and disinfected in accordance with the chapter „Cleaning and disinfection“. All components need to be cleaned after each use and all components must be disinfected before each use. Please wash hands thoroughly before touching the cleaned components.

1. Insert the valve (7) into the pump from below. Push it all the way.

2. Screw the feeding bottle (8) into the pump body (3) and tighten.

3. Insert the diaphragm (4) into the top part of the pump body (3). Press the diaphragm down to ensure it is stable.

4. Attach the cap (5) to the pump body (3). Connect one end of the tube (6) to the cap connector and the other end to the pump motor (10).

5. Insert the massage cushion (2) into the pump body (3). Press it down to ensure it fits properly. Connect the power adapter to the motor.

OPERATION INSTRUCTIONS

1. Press the ON/OFF button (starts in „stimulation“ mode automatically)
 2. Press the mode button (switch between „massage“ - „stimulation“ - „suction“ modes)
 3. Massage mode (9 levels)
 4. Stimulation mode (9 levels)
 5. Suction mode (9 levels)
- Each click on the „13“ and „14“ buttons will display the current level.
6. Working time of the pump is 30 minutes. After 30 minutes, the pump will shut down automatically.
 7. The feeding bottle should not be overfilled. Please follow indication line.
 8. 2 hours of charging will provide 90 minutes of working time.

PUMPING PROCEDURE INSTRUCTIONS

1. Wash hands thoroughly and ensure that your breast is clean. Relax and sit comfortably.
2. Place the breast into the pump, make sure the nipple is in the center.
3. Press “11” button to turn on the pump. Press “12” button to cycle between modes.

White - “massage” mode

Yellow - “stimulation” mode

Blue - “suction” mode

Press “13” and “14” buttons to increase/decrease power to match your comfort.

Note: You don't need to use the maximum suction, choose the suction that makes you feel comfortable.

4. After pumping, turn off the breast pump, and then carefully remove the breast from the breast pump.
5. Unscrew the feeding bottle from the breast pump to prepare for nursing or storage. To clean the used breast pump components, follow instruction in chapter „Cleaning and disinfection“.

CLEANING AND DISINFECTION

Warning! The motor unit should only be wiped down with a damp cloth.

Never wash the motor unit.

„Before first use, clean and disinfect all parts of the pump. Wash all the parts every time before use. The breast pump must be disassembled before every cleaning.“

Be careful when removing and cleaning the silicone valve “7”. If the valve is damaged, the breast pump will not work properly. To remove the valve, gently pull the sides of the valve. Clean all the components with hot water and a little dish washing soap and rinse thoroughly. To clean the valve, rub it gently, using warm water and dish washing soap. Do not insert any object into the valve, it might lead to damage. Alternatively, the components can be washed in the dishwasher. Place the parts in upper basket. Disinfect all the components except the by sterilizing for 5 minutes.

STORAGE AND HEATING OF BREAST MILK

Breast milk storage

After the breast milk is collected, close the container cover tightly or seal the storage bag, the milk must be put into the refrigerator immediately and can be refrigerated or frozen. Milk can be refrigerated for 3 days or frozen for 6-12 months. Never refreeze breast milk. Do not add fresh breast milk into frozen milk.

Breast milk heating

Use the milk/food warmer or warm water to thaw breast milk. Never use a microwave or boiling water to heat breast milk.

Attention

1. If you are freezing the breast milk, fill the container only to approximately two thirds of the capacity to leave space for volume expansion when freezing.
2. Don't place breast milk in the refrigerator door to avoid temperature fluctuations.
3. Gently shake the bottle to mix the contents.
5. Do not heat the frozen breast milk in a microwave or boiling water to avoid loss of nutrition and scalding.

Warning

This is a personal product, do not share it to avoiding health risks. When removing or connecting parts, operate carefully, do not overexert to avoid damage. This product should not be exposed to sun and should be kept away from children.

TROUBLESHOOTING

Lack suction power

Check whether the breast pump components are correctly assembled and the valve is not damaged. Make sure the massage cushion „2“, silicone diaphragm „3“ are firmly attached to the pump body to achieve ideal seal.

Unable to collect milk

Check the suction with your hand. Check the assembly of the pump. Try to relax when pumping. Repeat attempts with different settings.

Breast pain when pumping

Pain may be caused by excessive sucking force. You do not need to use the most powerful setting. Start over and try to find comfortable lower setting. You can also consult breast feeding specialist.

INFORMATION ABOUT DISPOSAL AND RECYCLING

This product is marked with the symbol for separate collection. The product must be disposed of in accordance with the regulations for disposal of electrical and electronic equipment (Directive 2012/19/EU on waste electrical and electronic equipment). Disposal together with regular municipal waste is prohibited. Dispose of all electrical and electronic products in accordance with all local and European regulations at the designated collection points which hold the appropriate authorisation and certification in line with the local and legislative regulations. Correct disposal and recycling help to minimise impacts on the environment and human health. Further information regarding disposal can be obtained from the vendor, authorised service centre or local authorities.

EU DECLARATION OF CONFORMITY

Hereby, Tesla Global Limited declares that the radio equipment type TSL-PC-BP100 is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: tsl.sh/doc

Manufacturer
Tesla Global Limited
Far East Consortium Building
121 Des Voeux Road Central
Hong Kong
www.teslasmart.com

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